

MULTI-PURPOSE TUBULAR BANDAGE

• Tubigrip provides tissue support in the treatment of strains and sprains, soft tissue injuries, general oedema and ribcage injuries.



ABOUT TUBIGRIP

Tubigrip provides lasting, effective support with complete freedom of movement for the patient. Once the bandage has been applied, covered elastic threads within the fabric move to adjust to the contours of the body and distribute pressure evenly over the surface. Fitting is quick and easy.

Tubigrip stays securely in position without pins or tapes, unlike conventional crepe bandages. As it can easily be reapplied after washing, without loss of effect, Tubigrip is particularly useful for outpatients and in the community.

Tubigrip reduces wastage, as only the exact amount required is cut from the roll. Apply with a 2–3cm overlap. Tubigrip is available in two shades – natural and beige.

TUBULAR BANDAGE APPLICATORS

Tubular Bandage Applicators may be used to make application easier over bulky dressings or painful joints.

Tubigrip is loaded onto the applicator, which is then positioned over the affected area. The end of the bandage is held in position while the applicator is gently removed, leaving the Tubigrip in place. Form a double layer by doubling back the bandage. Ensure that the upper edge of the second layer is taken 2–3cm higher up the limb than the first.

The applicators are available in a full range of sizes and are made from durable chrome-plated steel.

HOW TO APPLY TUBIGRIP



1. Cut Tubigrip to twice length required for limb, allowing an extra 2-3cm for overlap.

THE BENEFITS OF **USING TUBIGRIP**

- Provides comfortable, effective tissue support
- Easy to apply and reapply
- Full range of sizes to suit most applications •
- No pins or tapes needed
- Washable and reusable
- Choice of two colours for patient acceptability



2. Pull Tubigrip onto limb like a stocking.



3. Double Tubigrip back over limb. Ensure upper edge is taken 2–3cm higher up the limb than the first

APPLICAT	ORS SET	Art. No. 2953-00		
Art. No.	Applicator Size (Tubigrip Size)	PIP Code		
2913 2915 2916 2917 2918 2919 2919 2920	E (B,C) F (B,C,D) G (D,E,F) H (E,F,G) J (F,G) K (G,J) FX (C,D)	029-3936 029-3944 000-2634 031-9095 029-3951 000-8847 001-3383		

Tubigrip Sizing Guides are available to help you select the appropriate size and pressure for the condition being treated and can be ordered free from www.tubigrip.com

			A	В	С	D	E	F	G	J	K
	0.5m	Art. No.	-	1510	1513	1511	1514	1512	1515	-	-
a.		PIP Code	-	029-3407	023-8154	029-3415	023-8246	029-3423	023-8253	-	-
ğ		NHS Code	-	-	-	-	-	-	-	-	-
Shade	1m	Art. No.	-	1520	1521	1522	1528	1523	1524	-	-
n n		PIP Code	-	029-3456	029-3464	029-3472	048-9971	029-3480	029-3498	-	-
Natural		NHS Code	-	-	-	-	-	-	-	-	-
	10m	Art. No.	1435	1436	1443	1437	1434	1438	1439	1440	1441
		PIP Code	029-3258	029-3266	029-3282	029-3290	023-1969	029-3316	029-3324	029-3332	029-3340
		NHS Code	EGA011	EGA013	EGA015	EGA017	EGA019	EGA021	EGA023	EGA025	EGA027
ъ	1m	Art. No.	-	-	1545	1546	1547	1548	-	-	-
ad		PIP Code	-	-	006-8833	006-8973	006-9146	006-9153	-	-	-
Shade		NHS Code	-	-	-	-	-	-	-	-	-
Beige	10m	Art. No.	-	1449	1450	1451	1448	1452	1453	-	-
		PIP Code	_	029-3365	029-3373	029-3381	023-1977	029-3399	023-8386	-	-

KEY

- A Infant feet and arms
- **B** Small hands and arms
- С
- Medium arms, small ankles
 Large arms, medium ankles, small knees D Е
- Large ankles, medium knees, small thighs

F - large knees, medium thighs

- **G** Large thighs J - Large trunks
- K X-Large trunks



0800 7311 876 • info.uk@molnlycke.com • www.molnlycke.co.uk • www.tubigrip.com Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire, LU5 5UL, UK The Mölnlycke Health Care name and logo, Tubigrip™ are registered trademarks of Mölnlycke Health Care AB.©2010 Mölnlycke Health Care. All rights reserved.

