

# A SAFE AND CONNECTED APPROACH

  
TO ENDOSCOPE REPROCESSING







## The Revital-Ox™ Difference.

When you reprocess your endoscopy devices using the Revital-Ox system, you'll enjoy all the benefits of our noticeably superior chemistries:



**Greater Productivity**



**Improved Performance**



**Increased Safety**



**Green Savings**

## Get the Complete Revital-Ox™ System

STERIS has a range of endoscopy cleaning and disinfecting products and accessories that deliver peace-of-mind for hospitals, staff and patients. When you use the full range of Revital-Ox products, from bedside pre-cleaning, through high level disinfection, you get superior performance and an environmentally friendly solution for endoscope reprocessing.



## Pre-Cleaning & Transport

### Revital-Ox Bedside Complete®

Revital-Ox Bedside Complete is an all-in-one, pre-diluted enzymatic detergent and impregnated sponge designed for bedside pre-cleaning of flexible and rigid endoscopes, accessories and other medical devices.



#### SAFETY

- Free-rinsing solution
- Pre-diluted detergent eliminates manual mixing and dilution errors
- Convenient stand-up pouch helps to prevent spills



#### PRODUCTIVITY

- No mixing saves time



#### GREEN SAVINGS

- Sponge is made of 25% organic soy
- Enzymatic solution contains biodegradable ingredients

### Revital-Ox™ Container System

The Revital-Ox Container System is designed to safely transport and/or soak flexible and rigid endoscopes and accessories.

#### Revital-Ox Flexible Container System

- 20" diameter
- Safely holds coiled endoscopes

#### Revital-Ox Rigid Container System

- 21" x 13" x 6"
- Includes convenient drip tray for small devices and accessories, and minimizes direct contact with the solution

# Step 2:

## Revital-Ox Enzymatic Sponges

One-time use sponges that are impregnated with pre-diluted enzymatic detergent. Also available as a dry sponge.



## Cleaning

### Revital-Ox™ Enzymatic Detergents

Revital-Ox Enzymatic Detergents are low foaming, neutral pH, free rinsing, contain biodegradable ingredients and are 2x concentrated so you use less.



#### SAFETY

- Does not stain devices or clothing
- Free rinsing
- Dye- and fragrance-free options to address your staff's sensitivity/allergies



#### PRODUCTIVITY

- Equipped with PowerLift Cleaning Technology
- Quick soak times: 1-5 min.
- Cleans up to 4x faster than competitive detergents



#### GREEN SAVINGS


- Formulated with biodegradable ingredients
- Recycled packaging
- Concentrated formulation allows for less detergent usage per cycle

## The Revital-Ox Enzymatic Detergent Line Up

	 		
<b>DYE-FREE/FRAGRANT</b> <input type="checkbox"/> Color <input checked="" type="checkbox"/> Fragrance • Pleasant floral fragrance fights odors on contact • Dye-free for staff with sensitive skin	<b>DEEP CLEANING</b> <input checked="" type="checkbox"/> Color <input type="checkbox"/> Fragrance • Contains PowerLift Technology which is tough on difficult to remove soils • Teal color-enhanced for visibility in the sink	<b>FRAGRANT/ENHANCED WITH COLOR</b> <input checked="" type="checkbox"/> Color <input checked="" type="checkbox"/> Fragrance • Pleasant floral fragrance fights odors on contact • Purple color-enhanced for visibility in the sink	<b>DYE/FRAGRANT-FREE</b> <input type="checkbox"/> Color <input type="checkbox"/> Fragrance • Addresses patient and staff sensitivities and allergies • Superior fighting properties on bioburden and soils without added color and fragrance

\*Chemistry color shown in in-use dilution.

## The Revital-Ox™ Channel Brush Line Up

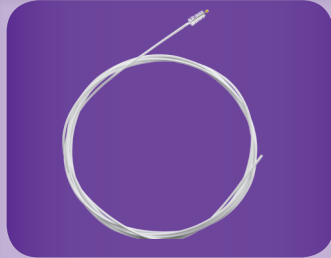


### Revital-Ox™ Single-Ended Channel Brush

A product dedicated to effectively cleaning endoscope channels



### Revital-Ox™ Dual- Ended Pull Through Channel Brush

Design allows you to push debris through with leading brush then remove remaining debris with trailing brush

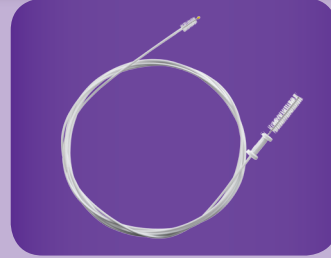

### Revital-Ox™ All-in- One Channel and Valve/Control Brush

Channel brush at one end and a valve control brush on the other end provides you a greater degree of flexibility when cleaning



### Revital-Ox™ Valve/ Control Brush

Helps you remove residual soils from control head and valve openings with an ergonomic design for a relaxed and balanced grip



## Revital-Ox™ Endoscopy Channel Brushes

Single-use cleaning brushes designed to fit your Endoscopic cleaning needs



### PERFORMANCE

- Easy to maneuver through complex lumens
- Pull through design prevents back and forth motion, pulling soils out with an easy motion



### SAFETY

- Non-abrasive nylon bristle, minimizes potential for scratching endoscopes
- Fluorescent rounded tips protect against scope damage and make it easy to see when immersed





# High Level Disinfection

## Revital-Ox™ RESERT® High Level Disinfectant

Revital-Ox RESERT High Level Disinfectant is a ready-to-use, fast, safe alternative to commonly used aldehyde [gluteraldehyde and ortho-phthalaldehyde (OPA)] solutions. An oxidative chemistry, Revital-Ox RESERT High Level Disinfectant uses only 2% Accelerated Hydrogen Peroxide (AHP), eliminating staff and patient exposure hazards associated with aldehyde-based High Level Disinfectant solutions. To help streamline your process, Revital-Ox RESERT High Level Disinfectant can be used for both manual soaking and automated endoscope reprocessors.



### SAFETY

- No extra ventilation required
- Leaves no toxic residues on devices
- No patient contraindications, contact dermatitis or respiratory distress



### PRODUCTIVITY

- One rinse, saving time and water
- 8-minute contact time
- Re-useable for up to 21 days



### GREEN SAVINGS

- Formulated with biodegradable ingredients
- Requires no detoxification – pours safely down the drain
- Single rinse requires less water than other high level disinfectants
- No heating required for activation



**RevitalOx**  
D Sponge

RevitalOx D Sponge is a sponge designed for pre-cleaning and removal of soil from the exterior of flexible endoscopes and other medical devices. Its unique porous matrix and enzymatic action, operating effectively pre-cleaning and pre-rinsing, and prevents soil from drying and proteinizing on the device. RevitalOx D Sponge is manufactured with 20% enzyme only.

**INSTRUCTIONS FOR USE**

- Remove sponge from package, dip in pre-cleaning solution, and squeeze sponge gently to remove solution, and place device and wipe down thoroughly with water.
- Place device with water.
- Place sponge on device and wipe down thoroughly with water.
- Remove sponge immediately after use - for single use only.

**STERIS**  
**RevitalOx**  
Enzymatic Sponge

RevitalOx Enzymatic Sponge is a sponge impregnated with an enzymatic solution to pre-clean and remove protein and lipid soils from the exterior of flexible endoscopes and other medical devices. Its unique porous matrix and enzymatic action, operating effectively pre-cleaning and pre-rinsing, and prevents soil from drying and proteinizing on the device. RevitalOx Enzymatic Sponge is manufactured with 20% enzyme only.

**INSTRUCTIONS FOR USE**

- Remove sponge from package and place around endoscope insertion tube or surgical instruments and wipe thoroughly with water.
- Place device with water.
- Place sponge immediately after use - for single use only.

**Caution:** Contains Substrate (polystyrene enzyme) and other components. Avoid contact with eyes, skin, mucous membranes, and clothing. If contact occurs, flush immediately with water. If contact occurs with eyes, flush with water for at least 15 minutes. Get medical attention. If contact occurs with skin, flush with water for at least 15 minutes. Get medical attention. If contact occurs with clothing, remove clothing and flush with water. If contact occurs with mucous membranes, flush with water for at least 15 minutes. Get medical attention. If contact occurs with mucous membranes, flush with water for at least 15 minutes. Get medical attention. If contact occurs with mucous membranes, flush with water for at least 15 minutes. Get medical attention.

**RevitalOx**  
Bedside Complete

RevitalOx Bedside Complete is a disinfectant solution designed for use on medical devices and surfaces. It is a broad-spectrum disinfectant that is effective against a wide range of microorganisms, including bacteria, viruses, and fungi. It is easy to use and safe for most surfaces.

**INSTRUCTIONS FOR USE**

- Apply disinfectant solution to the surface to be disinfected.
- Allow the solution to remain on the surface for the required contact time.
- Rinse the surface with water after disinfection.

**RevitalOx**  
RESERT  
HIGH LEVEL DISINFECTANT

RevitalOx RESERT is a high-level disinfectant solution designed for use on medical devices and surfaces. It is a broad-spectrum disinfectant that is effective against a wide range of microorganisms, including bacteria, viruses, and fungi. It is easy to use and safe for most surfaces.

**INSTRUCTIONS FOR USE**

- Apply disinfectant solution to the surface to be disinfected.
- Allow the solution to remain on the surface for the required contact time.
- Rinse the surface with water after disinfection.

**RevitalOx**  
ENZYMATIC 2X  
DISINFECTANT

RevitalOx ENZYMATIC 2X is a broad-spectrum disinfectant solution designed for use on medical devices and surfaces. It is a broad-spectrum disinfectant that is effective against a wide range of microorganisms, including bacteria, viruses, and fungi. It is easy to use and safe for most surfaces.

**INSTRUCTIONS FOR USE**

- Apply disinfectant solution to the surface to be disinfected.
- Allow the solution to remain on the surface for the required contact time.
- Rinse the surface with water after disinfection.

**RevitalOx**  
ENZYMATIC 2X  
DISINFECTANT

RevitalOx ENZYMATIC 2X is a broad-spectrum disinfectant solution designed for use on medical devices and surfaces. It is a broad-spectrum disinfectant that is effective against a wide range of microorganisms, including bacteria, viruses, and fungi. It is easy to use and safe for most surfaces.

**INSTRUCTIONS FOR USE**

- Apply disinfectant solution to the surface to be disinfected.
- Allow the solution to remain on the surface for the required contact time.
- Rinse the surface with water after disinfection.

**RevitalOx**  
ENZYMATIC 2X  
DISINFECTANT

RevitalOx ENZYMATIC 2X is a broad-spectrum disinfectant solution designed for use on medical devices and surfaces. It is a broad-spectrum disinfectant that is effective against a wide range of microorganisms, including bacteria, viruses, and fungi. It is easy to use and safe for most surfaces.

**INSTRUCTIONS FOR USE**

- Apply disinfectant solution to the surface to be disinfected.
- Allow the solution to remain on the surface for the required contact time.
- Rinse the surface with water after disinfection.



## Ordering Information

	SKU	Product Name	Size	Pack
<b>Step 1:</b>	2D91QX	Revital-Ox™ Bedside Complete®	215ml	35/case
	2D8850WR	Revital-Ox™ Bedside Complete®	500ml	25/case
	2D93Q0	Revital-Ox™ Rigid Container System	n/a	1
	2D94Q0	Revital-Ox™ Flexible Container System	n/a	1
<b>Step 2:</b>	2D99AW	Revital-Ox™ Enzymatic Detergent (with Fragrance)	4 liters	4
	2D96AW	Revital-Ox™ Enzymatic Detergent (with Color and Fragrance)	4 liters	4
	2D97AW	Revital-Ox™ Dye & Fragrance Free Enzymatic Detergent	4 liters	4
	2D98AW	Revital-Ox™ PowerLift Cleaning Technology Enzymatic Detergent	4 liters	4
	2D90QH	Revital-Ox™ Enzymatic Sponge	4	25
	2D92QH	Revital-Ox™ D Sponge	4	25
			<b>Length/ Channel</b>	<b>Pack</b>
	2D75QV	Revital-Ox™ Single-ended Channel Brush	230cm/2.0+mm	50/case
	2D76QH	Revital-Ox™ Dual-ended Pull Through Channel Brush	230cm/2.0+mm	100/case
	2D77QH	Revital-Ox™ All-in-One Channel & Valve/Control Brush	230cm/2.0+mm	100/case
	2D78QV	Revital-Ox™ Valve/Control Brush	n/a	50/case
<b>Step 3:</b>	4455AW	Revital-Ox™ RESERT® High Level Disinfectant	4 liters	4
	4455N9	Revital-Ox™ RESERT® High Level Disinfectant	1 liter	6
	PCC053	Revital-Ox™ RESERT™ R60 Solution Test Strips	60 strips	1/box
	PCC054	Revital-Ox™ RESERT™ R60 Solution Test Strips	60 strips	2/box



Scan this code or visit [www.steris.com/revitalox](http://www.steris.com/revitalox) to learn more.



STERIS Corporation  
5960 Heisley Road  
Mentor, OH 44060-1834 · USA  
440-354-2600 · 800-548-4873  
[www.steris.com](http://www.steris.com)