


Clorox Commercial Solutions® Clorox® Total 360® Disinfectant Cleaner₁



1. Connect Clorox Commercial Solutions® Clorox® Total 360® Disinfectant Cleaner₁ bottle to Clorox® Total 360® Electrostatic Sprayer. Put on appropriate PPE.*

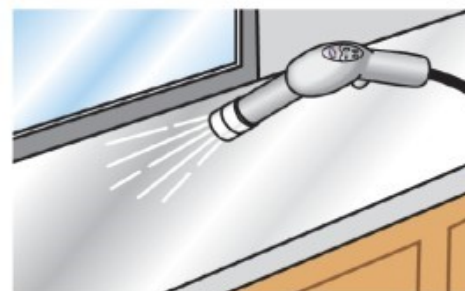

2. Spray surfaces using a sweeping back-and-forth motion until covered with the product. Allow surfaces to remain visibly wet for 2 minutes.



3. Allow treated surfaces to air dry.



*Use of fully enclosed eyewear protection, such as goggles, is recommended when using the Clorox® Total 360® Electrostatic Sprayer. Wear NIOSH/OSHA approved respiratory protection when using through the Clorox® Total 360® Electrostatic Sprayer: N95 respirator. Respiratory protection must be provided in accordance with current local regulation.

