



MetriCide OPA Plus is the ideal choice for healthcare professionals who are concerned about the cost and effectiveness of high-level disinfectants used in reprocessing endoscopes and other heat sensitive semi-critical medical devices.

Metrex™ is committed to product innovation and improvement.
Now, MetriCide OPA Plus can be reused for up to 30 days manually.

MetriCide OPA Plus Features:

- Manual reprocessing: can be reused for up to 30 days in 12 minutes at 20°C – No other OPA lasts longer manually
- Automated reprocessing (AER): can be reused for up to 14 days in 5 minutes at 25°C
- Surfactant free formula – won't cloud scope lenses
- Gentle on flexible endoscopes and other surgical instruments
- MetriCide OPA Plus Test Strips read 33% faster than other OPA brands*



For more information on how you can get started with MetriCide OPA Plus, contact us at **800.841.1428** today!

Metrex™
Protecting People

©Metrex Research, LLC. All Rights Reserved.

METRICIDE OPA PLUS SOLUTION

EFFICACY

Microbicidal Activity: The following table indicates the spectrum of activity as demonstrated by testing of MetriCide OPA Plus using prescribed test methods. Bactericidal, Fungicidal, Tuberculocidal Tests all passed in 5 minutes at 25°C.

MICROORGANISM

VEGETATIVE ORGANISMS

Staphylococcus aureus
Salmonella choleraesuis
Pseudomonas aeruginosa
Mycobacterium tuberculosis var: bovis

FUNGI

Trichophyton mentagrophytes

VIRUSES

NON-ENVELOPED

Poliovirus Type 1
 Rhinovirus Type 42
 Hepatitis A (AER)

ENVELOPED

Influenza Virus Type A [Hong Kong]
 HIV-1
 Herpes simplex Type 1
 Duck Hepatitis B
 Avian Influenza A



ORDERING INFORMATION

Part #	Product Description	Packaging
10-6000	MetriCide OPA Plus Solution 1 gallon	4 bottles/case
10-602	MetriCide OPA Plus Solution Test Strips - 100 Strips per bottle	2 bottles/case

Trust the company thousands of healthcare facilities use every day... Trust Metrex for all your instrument reprocessing needs.

