

DENTAL UNIT

## WATERLINE MAINTENANCE TABLETS

750 mL  
50 Tablets

2 L  
50 Tablets

PREP

### Directions for Use

#### Initial Treatment

Before starting a treatment program with Henry Schein Waterline Maintenance Tablets, shock lines with a waterline cleaner or another product recommended by the equipment manufacturer to remove line buildup or deposits. When shocking dental unit waterlines, always follow a recognized line shocking/cleaning protocol. Repeat shocking to clean lines every 2 months or more frequently when water quality results indicate.

#### Routine Treatment

1. Place tablet or tablets (see table below) in empty dental unit water bottle every time you refill the water. Henry Schein Waterline Maintenance Tablets are intended for use with potable water (including tap or distilled).
2. Fill bottle with desired volume of water and connect to dental unit. Tablet dissolves in a few minutes.

TREAT

750 mL Tablets		2 L Tablets	
WATERLINE VOLUME	TABLETS	WATERLINE VOLUME	TABLETS
700-750 mL (24-25 fl. oz.)	1	2 L (68 fl. oz.)	1
1500 mL (51 fl. oz.)	2	4 L (135 fl. oz.)	2
2000 mL (68 fl. oz.)	3		

3. Henry Schein Waterline Maintenance Tablets may be left in the waterlines overnight and for extended periods of non-use. Use water solution until it is expended or up to 28 days. It is not necessary to purge lines at the end of each day.

TEST

#### Waterline Testing

Clinical monitoring of water quality can ensure that procedures are being performed correctly and are effective. Test water quality quarterly to establish effectiveness of your maintenance program. If water quality results approach or exceed requirements, repeat Initial Treatment.



### Henry Schein® WATERLINE MAINTENANCE TABLETS

2 L 570-2560  
750 mL 570-2561