

CRITERION™ NITRILE EXAM N200 GLOVES

ROBUST PROTECTION IN STYLISH BLACK



PLUS, NOW FENTANYL-TESTED
Exclusive to New Nitrile Exam N200 & N300 Black Gloves
(SEE BACK FOR MORE INFORMATION)



For more information about the full line of Criterion Personal Protective Equipment, scan here or visit HENRYSCHEIN.COM/CRITERION

CRITERION™ NITRILE EXAM N200 GLOVES

- Durable nitrile construction, slightly thinner than the Criterion™ Nitrile Exam N100 Gloves for instances where dexterity is important
- Chlorinated for reduced friction and easy donning
- Textured fingers provide enhanced grip
- Beaded cuff. 2.7-mil/0.07mm thickness at palm



NEW!

NOW IN BLACK

CHEMOTHERAPY TESTED TO ASTM D6978 STANDARD

Caution: gloves used for protection against chemotherapy drug exposure must be selected specifically for the type of chemicals used. This glove has been tested for resistance to permeation of various chemotherapy drugs per ASTM D6978, "standard practice for assessment of resistance of medical gloves to permeation by chemotherapy drugs."

Warning: Do not use Carmustine (BCNU) (3.3mg/ml) and Thiotepea (10mg/ml) When chemotherapy drugs are present, gloves selection should be based on the specific types of drugs used. Users should review drug labeling or material safety data sheets for the drugs being used to determine an adequate level of protection.

FENTANYL-TESTED

Tested using ASTM D6978 test method, which evaluates glove permeation resistance to fentanyl. The results show that gloves have resistance to fentanyl for at least 240 minutes.

Fentanyl breakthrough time (100 mcg/2ml): >240 minutes

CHEMOTHERAPY DRUGS & CONCENTRATION

MINIMUM BREAKTHROUGH DETECTION TIME

Carmustine (BCNU) (3.3mg/ml)	Do Not Use
Cisplatin (1.0mg/ml)	>240min
Cyclophosphamide (Cytoxan) (20mg/ml)	>240min
Dacarbazine (DTIC) (10.0mg/ml)	>240min
Doxorubicin Hydrochloride (2.0mg/ml)	>240min
Etoposide (Toposar) (20.0mg/ml)	>240min
Fluorouracil (50.0mg/ml)	>240min
Methotrexate (25mg/ml)	>240min
Mitomycin C (0.5mg/ml)	>240min
Paclitaxel (Taxol) (6.0mg/ml)	>240min
Thiotepea (10.0mg/ml)	Do Not Use
Vincristine Sulfate (1.0mg/ml)	>240min

BLACK GLOVES

X-Small 200/box 570-3487

Large 200/box 570-3490

Small 200/box 570-3488

X-Large 180/box 570-3491

Medium 200/box 570-3489

SHOP THE FULL HEAD-TO-TOE PROTECTION IN STYLISH BLACK




Temple Grip Protective Eyewear



Earloop Masks Levels 1 & 3



Nitrile Exam N200 & N300 Gloves

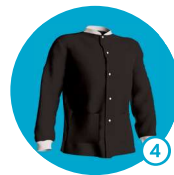

SMS Jackets and Lab Coats

